


Fitting NEO rear wheels with disc fitted. Part number 1021-DB

1. Remove the calliper from mounting plate with a 4mm allen key, ensure you don't lose the washers between the plate and the calliper - you will need these when you replace.

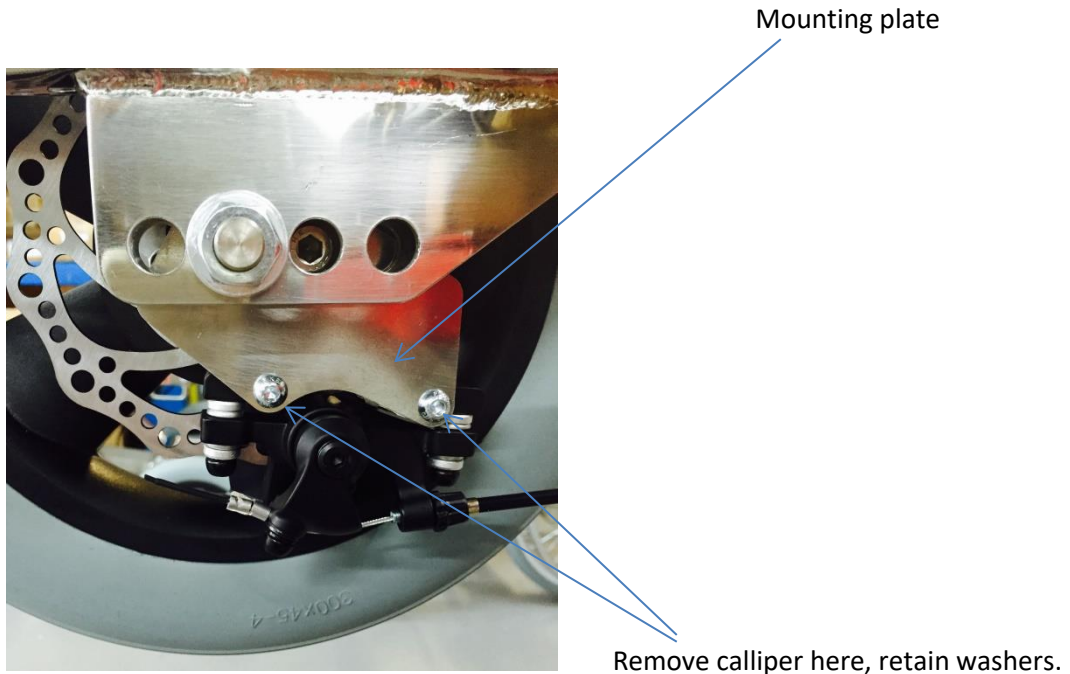

2. Remove the outer black hub cap to reveal and then remove the 19mm external wheel nut.
3. Withdraw old wheel complete with disc and fit new wheel and disc on existing axle, ensuring all spacers are returned as original. Note disc is floating and will move on M5 studs.
4. Re-attach the calliper to the mounting plate; ensure the disc fits between the disc pads. Replace the washers as per point 1, as leaving them out will make the brakes bind.
5. Replace the external wheel nut – add some lock tight and re-tighten.
6. Replace the black outer hub cap.
7. For adjustment of the callipers see NEO Disc Brake adjustment sheet. Available from Southwest Seating & Rehab Ltd.